
	<b>Respirator Mask FFP-3</b>	
	<b>Application Area and Properties</b>	

- **Features:** Half-mask with a high filtering capacity, tight-fitting and leakproof. The soft material and special shape ensure that it can be easily adjusted to fit any face shape. Minimal respiratory resistance and the exhalation valve make it extremely comfortable to wear. The threaded elastic strap makes it easy to put on and take off. Type tested and certified in accordance with EC directive 89/686/EEC (Personal protective equipment).
- **Area of application:** Protection from solid and liquid particles, e.g. for handling CMR<sup>1)</sup> drugs (e.g. cytostatics, virostatics, biological agents and viruses).
- **Protection capacity:** Protection from all CMR drugs or chemicals is not guaranteed. Does not protect against gases and vapours!
- **Replacement interval:** Depends on inhalation resistance. Maximum use for one shift (8 hours) with at least three breaks in between. For use with CMR-drugs, a maximum of 4 hrs, so far as there is no obvious contamination. Replace immediately in the event of visible contamination.  
**Single use mask – Do not re-use respirator mask!**
- **Before use:** Check for damage. Do not use damaged masks.
- **Limitation:** Single use mask! Laboratory test results can differ from values obtained in practice considerably and, as a result, the useful life of the mask may increase or decrease. Facial hair and scars can impair the fit of the mask and therefore the level of protection.
- **Disposal:** If used in combination with CMR-drugs and contaminated areas/items: waste requiring supervision (waste code: 18 01 04 in accordance with 2000/532/EC); in the event of heavy contamination, waste requiring special supervision (waste code: 18 01 08\* in accordance with 2000/532/EC); collect and dispose of waste separately!

<sup>1)</sup>: Carcinogenic; mutagenic; toxic to reproduction;

### Types




Size	Universal	
Item no.	121285	121286
	non-sterile	sterile
Packaging unit (PU)	1 piece	4 pieces
Size of single pouch (PU)	125 mm x 135 mm	

### Material Properties

Latex free	Yes, complete mask
	Several filter material layers
Weight	Excl. packaging : 18 g

### Particle Filtration Efficiency

Test aerosol	Minimum filtration efficiency
Sodium chloride	> 99%
Paraffin oil	> 99%
Dolomite clogging test (for FFP3-mask) (Dolomite DRB4/15b test aerosol)	Passed
Gases	Unsuitable

	<b>Sterilisation</b>
<b>Process</b>	Fumigation with ethylene oxide
<b>CE-Mark</b>	
<p>CE-mark for <b>complex PPE in category III</b> in accordance with <b>PPE Directive 89/686/EEC</b>. The <b>type test</b> performed was based on EN 149:2001+A1:2009. Documented by the <b>EC type test certificate</b> of the manufacturer. <b>Quality assurance</b> (EC quality assurance system with monitoring): Control measures by intermediary notified body CIOPIB (1437) in accordance with 89/686/EEC.</p>	
<b>Notified Body „1437“</b>	
<p><b>CIOPIB (The Central Institute for Labour Protection – National Research Institute) Testing, monitoring and certification body No. 1437</b>, Czerniakowska 16, 00-701 Warsaw, Poland.</p>	
	<b>Quality Management System</b>
<p>Our <b>quality management system</b> is <b>tested and certified</b> by TÜV Product Service GmbH (a certification body accredited by the German Accreditation Council) in accordance with DIN EN ISO 9001:2008. Regular <b>audits and production site inspections</b> guarantee the quality of our products.</p>	
<b>Storage and Transport Conditions</b>	
<ul style="list-style-type: none"> <li>▪ Dark (protect from direct UV light and sunlight)</li> <li>▪ Cool (-20 to +40°C)</li> <li>▪ Dry</li> <li>▪ No contact with pointed and/or sharp objects</li> </ul>	
	<b>Shelf Life</b>
<ul style="list-style-type: none"> <li>▪ 3 years</li> </ul>	
<b>Manufacturer</b>	
<p>UVEX Arbeitsschutz GmbH, Würzburger Str. 181 – 189, 90766 Fürth, Germany</p>	
<b>Distributor</b>	
<p>BERNER INTERNATIONAL GmbH Mühlenkamp 6 – D- 25337 Elmshorn Tel: 04121 4356 0 - Fax: 04121 4356 20 info@berner-international.de www.berner-international.eu</p>	